Surgical containers

Recipientes cirúrgicos

FR Conteneurs à usage chirurgical

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NL Chirurgische containers

IT Contenitori chirurgici

ES Contenedores quirúrgico

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EN Surgical containers

DEVICES

-Surgical bowls; basins and rectangular basins);
-Trays (includes kidney trays, catheterization trays, laparoscopy/endoscopy trays and trays with guidewire bowl);
-Basins (includes ring basins, placenta basins and rectangular basins);
-Basin liners;
-Pitchers;
-Lids (includes surgical bowl lids and basin lids).

DEVICE INFORMATION

Wide-range of single-use surgical containers comply with the highest quality standards. Medical devices classified as Class IIa devices due to their broad approved indications for use, including use during invasive procedures. Some references present an appropriate lid as well as adhesives for fixation to surfaces. Healthcare professionals should select a device with a suitable capacity and design appropriate for the intended purpose.

INTENDED PURPOSE AND INDICATIONS

Surgical bowls, trays, basins, basin liners and pitchers:

-To temporarily hold organs or tissues.
-To hold or for preparation of medical devices such as catheters with medicinal solutions.
-To hold aqueous, saline solutions for irrigation.
-To hold aqueous, saline or antiseptics solutions for wound cleansing.
-To hold antiseptics for disinfection of skin or mucosa.

Additionally, and due to their design, are intended to be used:
Pitchers - to pour solutions.

Kidney trays, catheterization trays (boat shape) - to fit the human body; are adequate to collect body fluids.

Trays with guidewire bowls - to prepare and to temporary store a guidewire while keeping it in place and hydrated.

Basin liners - The basin liner, designed with an integrated drape, is used to prevent the infectious agents transfer in a surgical context. Shaped to fit the interior of a ring basin, the liner includes a drape that covers the legs of the ring stand. Additionally, it serves to hold warm irrigation solutions.

Laparoscopy/Endoscopy trays - may be used with trocars.

Surgical bowl lids and Basin lids:

-To obtain the closed version of surgical bowls or basins during transport, storage or disposal.

Placenta basins:

-Specifically to temporarily hold placenta and/or umbilical cord for inspection after birth and for preparation of placental membrane which can after be used in a context such as woundcare.

WARNINGS AND PRECAUTIONS

-Not intended to be used for preparation of injectables nor bone cement.
-Visually inspect the sterile barrier system prior to use. Do not use the device if you detect any breach in packaging integrity.
-Visually inspect the device. Do not use the device if you detect any sign of deterioration or defect that may compromise the performance.
-Before use, check visually for particulate matter and use only if the contents are clear.
-For ophthalmic invasive procedures, flush the surgical container with sterile solution before use.
-Intended to be used exclusively by healthcare professionals.
-Do not re-use. Intended to be used one single time to prevent risks from contamination, including possible infections, and/or functional damage.
-Dispose according to applicable legislation and/or infection prevention standards for healthcare waste, to minimize potential risk of contamination, infection or pollution.

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